

# INSTALLATION, OPERATION AND MAINTENANCE INSTRUCTIONS





**RBA4007-122**

Drop Down Grab Rail

**RBA4007-123**

Drop Down Grab Rail with Toilet Roll Holder

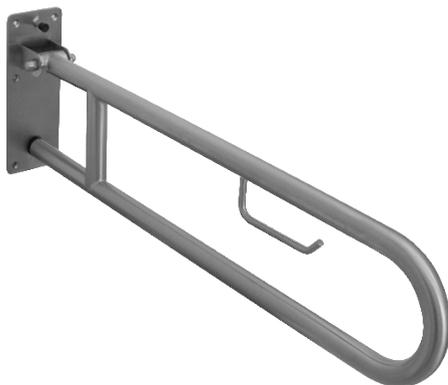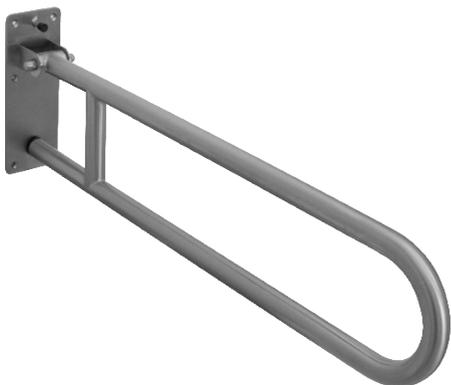

Model	Drop Down Rail
Date	15/05/19

Supersedes all previous

**SYDNEY | MELBOURNE | BRISBANE | PERTH | AUCKLAND**

**Australia** 1300 788 778 [www.rba.com.au](http://www.rba.com.au) | **New Zealand** 0800 722 111 [www.rbagroup.co.nz](http://www.rbagroup.co.nz)

## Description

Drop down grab rail, fabricated from 32mm [Dia] type 304 stainless steel. All exposed surfaces polished to a satin finish, and corners ground to a rounded surface.

## Components Supplied

Drop down grab rail  
6 x wall plugs  
6 x Screws

## Pre-Installation

1. Measure site to ensure correct size and location. Where required, ensure unit is installed to meet local building codes and specifiers instructions.
2. Ensure wall construction in fixing locations are able to support the unit, and the loads it will be subjected too.
3. This unit should be installed by suitably qualified persons, in a fit for purpose application, with suitable materials where required.

## Installation

1. Place unit on wall in location as required and mark up mounting location with each fixture.
2. Secure unit to wall at all 6 mounting points using suitable fixings.
3. Check to make sure the rail sits horizontal in the down position and that the mounting flange is flat against the wall.
4. Check to see if rail moves freely from horizontal to vertical position, and stays in the vertical position. Apply a small amount of force to release from vertical position and slowly lower back to horizontal position.
5. The drop down grab rail can be folded up when not in use.


## Operation & Maintenance

The rail can be lowered to the horizontal position by applying a small force to the rail to lower it. The rail can sit in the horizontal position to provide assistance to the user. The rail can then be folded up back into the vertical position when not in use.



Unit should be inspected regularly to ensure correct function of the drop down arm mechanism, no deformation of the structure and general wear and tear to prevent any issues that may affect performance or safety. Hinge-bolt may be tightened to ensure rail stays in the upright position when required.

# INSTALLATION, OPERATION AND MAINTENANCE INSTRUCTIONS

## Dimensions & Rough In

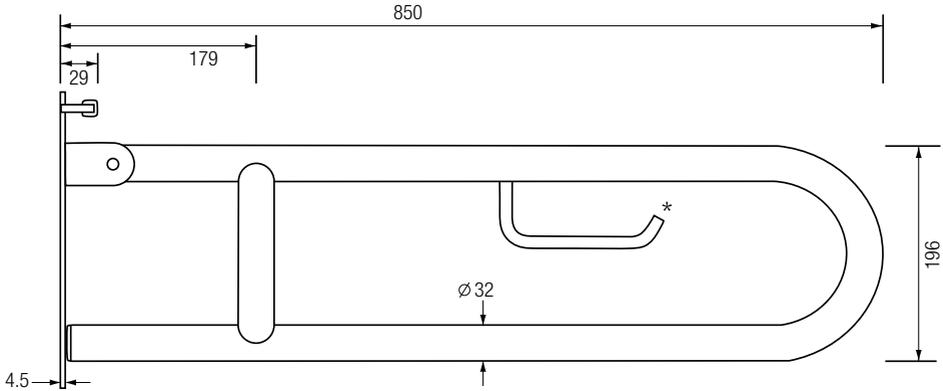

\* Toilet Roll Holder optional

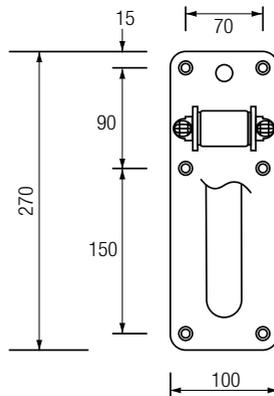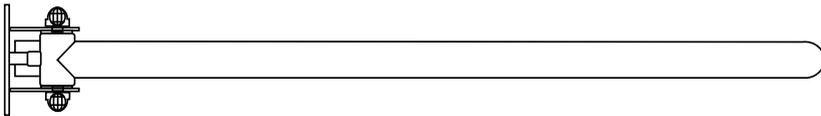

**Warning:** Grab rails are no stronger than the anchors or walls to which they are attached and must be firmly secured in order to support the loads for which they are intended.

**Important:** Installation Instructions are subject to change without notice. Please visit [www.rba.com.au](http://www.rba.com.au) for latest revision. Dimensions are subject to manufacturer's tolerance of +/-10mm. Rough-in should be completed with each fixture.

