

Note: This product should be installed, by suitably qualified persons, in a fit for purpose application, to suitable materials, using suitable fixings and comply with any relevant codes. It should be inspected periodically for signs of wear and tear that may affect performance or safety.

Dimensions are subject to manufacturer's tolerance of +/-10mm. Rough-in should be completed with each fixture.



RBA4157-777-005

Folding Shower Seat with Legs to Suit Changing Places
Design Specifications 2020



Model	RBA4157-777-005
Date	13/05/20
Doc#	ii/rba4157-777-005/0520

Supersedes all previous

PRIOR TO INSTALLATION

All folding shower seat installations, where support legs go to the Finished Floor Level [FFL], are unique due to the ‘fall’ of each individual floor. To ensure accurate installation, ensure the seat is level and the legs fully extended to meet the FFL before securing unit to the wall.

Warning: Folding Shower Seats are no stronger than the anchors or walls to which they are attached and, therefore, must be firmly secured in order to support the loads for which they are intended.

INSTALLATION

RBA4157-777-005 Folding Shower Seat meets the requirements of Changing Places Design Specifications 2020 when installed correctly.

1. Locate seat horizontally on wall to comply with Changing Places Design Specifications 2020. Secure unit to wall at mounting points shown on dimensional drawing.
2. Check to make sure seat surface is horizontal and frame is flat against wall.
3. Check to see if seat moves freely from horizontal to vertical position.
4. Shower seat can be folded up when not in use.

OPERATION

Legs lock into place to by pushing the legs into the wall to prevent seat from falling down when not in use.

To place seat back down from upright position pull both legs out and slowly lower seat to horizontal position

