

## MODEL      Folding Shower Seat

**RBA4157-777-003**      Folding Shower Seat with Legs



As improvements in the design and performance of RBA products are continuous, specifications may be subject to change without notice. The illustrations and descriptions herein are applicable to production as of the date of this Installation Instructions Sheet. Revised 12/22 © 2022 by RBA Group II/Model RBA4157-777-003/1222

## Prior to Installation

All folding shower seat installations, where support legs go to the Finished Floor Level [FFL], are unique due to the 'fall' of each individual floor. To ensure accurate installation, ensure the seat is level and the legs fully extended to meet the FFL before securing unit to the wall. Warning: Folding Shower Seats are no stronger than the anchors or walls to which they are attached and, therefore, must be firmly secured in order to support the loads for which they are intended.


## Installation

RBA Folding Shower Seats meet the requirements of AS1428.1 when installed correctly.

1. Locate seat horizontally on wall to comply with AS1428.1. Secure unit to wall at mounting points shown on dimensional drawing.
2. Check to make sure seat surface is horizontal and frame is flat against wall.
3. Check to see if seat moves freely from horizontal to vertical position.
4. Shower seat can be folded up when not in use.

## Operation

Legs lock into place to by pushing the legs into the wall to prevent seat from falling down when not in use. To place seat back down from upright position pull both legs out and slowly lower seat to horizontal position

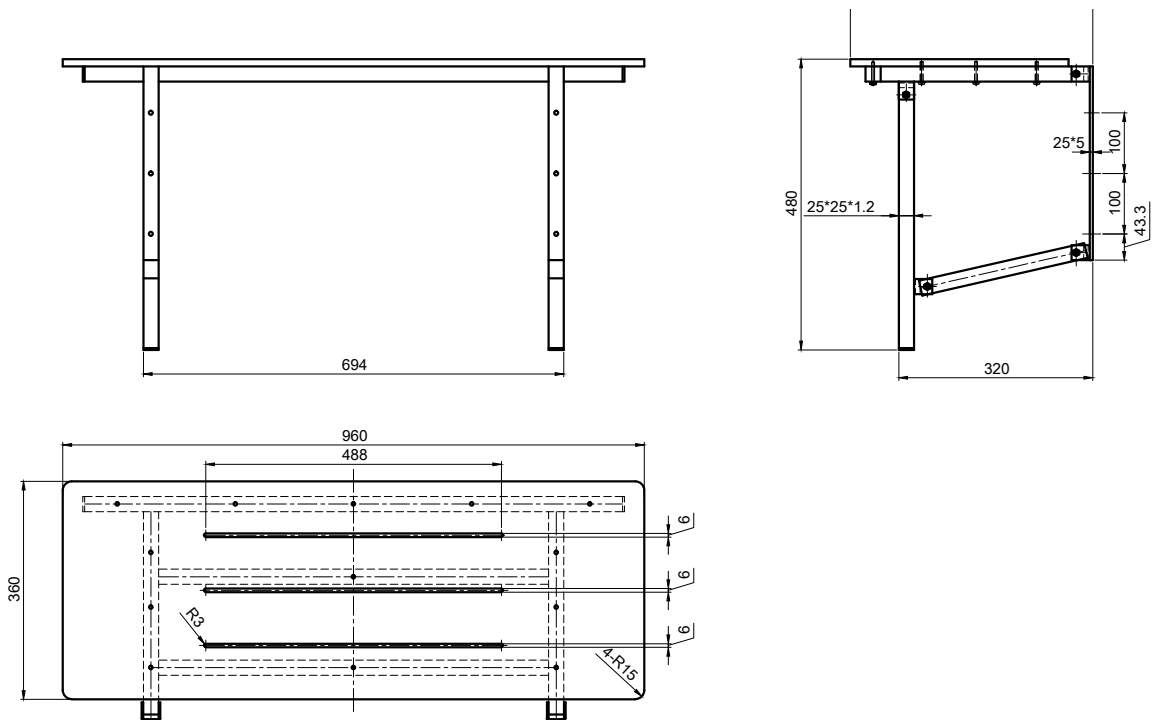

**Note:** This product should be installed, by suitably qualified persons, in a fit for purpose application, to suitable materials, using suitable fixings and comply with any relevant codes. It should be inspected periodically for signs of wear and tear that may affect performance or safety. Dimensions are subject to manufacturer's tolerance of +/-10mm.

Rough-in should be completed with each fixture.

**Important:** Installation Instructions are subject to change without notice.

Please visit [www.rba.com.au](http://www.rba.com.au) for latest revision.