


KB3000-AHL

Adjustable Height Changing Station

Koala Kare™

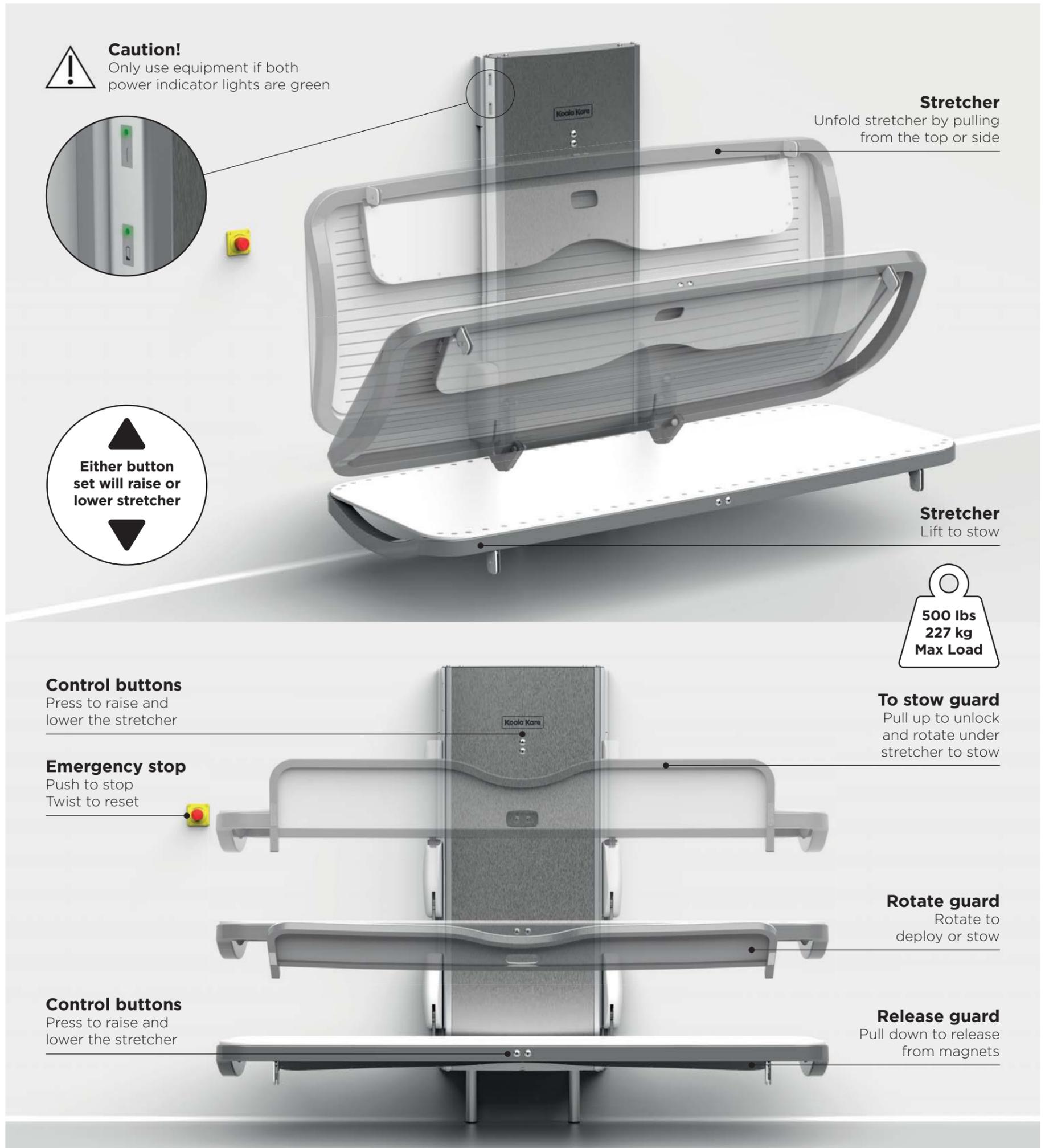

Caution!
Only use equipment if both power indicator lights are green

Stretcher
Unfold stretcher by pulling from the top or side

Either button set will raise or lower stretcher

Stretcher
Lift to stow

500 lbs
227 kg
Max Load

Control buttons
Press to raise and lower the stretcher

To stow guard
Pull up to unlock and rotate under stretcher to stow

Emergency stop
Push to stop
Twist to reset

Rotate guard
Rotate to deploy or stow

Control buttons
Press to raise and lower the stretcher

Release guard
Pull down to release from magnets

Warning! Risk of injury by crushing. Ensure the area underneath the stretcher is kept clear of obstructions before operating. Keep clear of the area above the stretcher when stowing.
Risk of falling. Stretcher occupant must not be left unattended.

Caution! Keep clear of moving parts when operating. Ensure there are no obstructions when moving the guard into position.

Action! Always deploy the guard before moving the changing table with a person on the stretcher.

This KB3000-AHL will require maintenance to ensure it is safe to operate. This service must be carried out by an approved, trained professional.



RBA Group, Koala Kare Authorised Distributor
Level 1, 32 Frederick Street, Oatley NSW 2223

Phone: 1300 788 778 | **Website:** rba.com.au | **Email:** sales@rba.com.au

Revised 03/26 by RBA Group

The KB3000 changing station should be regularly cleaned and must be cleaned between users to prevent the risk of cross contamination of infection.



CAUTION: The heavy use of oil based products may adversely affect the stretcher material. Oils and other petrochemical products should be avoided and any residue removed immediately. These products include, amongst others: baby oil and emollient creams.

- Before cleaning, try a small test first in an inconspicuous area with the cleaning agent you are proposing to use.
- The changing station may be cleaned using mild cleaning agents, surface disinfectants or with a warm soapy water solution. The changing station should be rinsed off after cleaning and dried off with a soft cloth.
- A soft cloth or sponge is recommended for cleaning the equipment, abrasive pads should not be used.
- Chemical agents, abrasive compounds and solvents must not be used, i.e. drain cleaners, acetone, oven cleaners or acids.
- Attention should always be paid to instruction labels for the cleaning agent.
- Do not use an autoclave or other type of steam cleaner to clean any part of the product.
- The main unit of the device may be cleaned with the same solution used on the stretcher.



ACTION: Ensure any cleaning product spillages are cleaned up correctly.



CAUTION: Always follow the instructions and comply with the safe use of the cleaning/disinfecting agent. Do not mix cleaning products.



CAUTION: Do not use flammable cleaning agents.



CAUTION: Do not use abrasive cloths or cleaners.



CAUTION: A dampened, non-wet cloth should be used for cleaning.



CAUTION: Do not pour or spray cleaning liquids directly onto the equipment.